



從陰影區域採集血樣。



合格

圓圈填滿血液並完全飽和

不合格

連續重疊採血



血量不足，多滴血



出現血清環



請填妥採血卡上所有資料。
若非以電子方式送檢，亦需填寫《生化遺傳學送檢單》
(用品編號 T439)。

採血卡填寫說明

腳跟（用於嬰兒）或手指（用於兒童及成人）針刺採血



MAYO CLINIC | Mayo Medical Laboratories
3050 Superior Drive NW | Rochester, MN 55901
MayoMedicalLaboratories.com | 800-533-1710

©2016 Mayo Foundation for Medical Education and Research. All rights reserved. MAYO, MAYO CLINIC, Mayo Medical Laboratories and the triple-shield Mayo logo are trademarks and service marks of MFMER.

T800
MC4091-67CH

